	Title	Instructions for Use Barrier Cream	
	Revision	03	
	Technical Documentation Number	TF010	

1.0 PRODUCT DESCRIPTION

Barrier Cream is a white concentrated cream that provides protection to skin from body fluids. It protects dry, chafed, red, or irritated skin by moisturising the skin and providing a long lasting barrier. Use of this cream allows adhesive products to stick to the skin.

2.0 INTENDED PURPOSE

Barrier Cream acts as a mechanical barrier, protects at-risk skin areas from damage and irritation associated with incontinence.

2.1 INTENDED POPULATION

The Barrier Cream may be used on adults and children

2.2 INTENDED USER

Intended for use by health professionals and caregivers and may be used in a hospital, community and home setting.

3.0 INDICATIONS

Barrier Cream protects at-risk skin areas from damage and irritation associated with incontinence. Barrier Cream maintains and preserves skin from damage associated with chronic incontinence. It moisturizes and protects severely dry skin. The cream will not reduce adhesion of medical tapes or adhesive dressings.

4.0 CONTRAINDICATIONS/ SAFETY INFORMATION

Barrier Cream should not be used on infected skin. Barrier Cream should not be used where there is known allergic sensitivity to the ingredients.

5.0 WARNINGS


Barrier Cream may increase the adherence of some adhesive products. If skin condition worsens or does not improve within 7 days, consult a physician. Keep out of reach of children. In case of ingestion, seek medical help. For external use only. Avoid contact with eyes.

6.0 INFORMATION FOR USE

Gently cleanse skin area as required. Spread Barrier Cream thinly to cover entire affected area. Repeat as necessary. If the feel is oily, too much has been applied.

Ingredients

Purified Water, Ethylhexyl Isononanoate, Disiloxane, Polyisobutylene, VP Eicosene Copolymer, Hydroxyethyl Acrylate / Sodium Acryloyl Dimethyl Taurate Copolymer, Butylene Glycol, Oleth-3 Phosphate, Phenoxyethanol, Hydrogenated Styrene/Isoprene Copolymer, Allantoin, Carbomer, Triethanolamine, Methylparaben, Propylparaben, Disodium EDTA

	Title	Instructions for Use Barrier Cream	
	Revision	03	
	Technical Documentation Number	TF010	

7.0 CLINICAL BENEFITS/ SPECIAL NOTES

The Barrier Cream forms a durable barrier that works as a protective shield between the skin and bodily wastes and fluids. The product is designed to assist barrier resistance against irritation from body fluids.

8.0 STORAGE




Store dressings away from direct sunlight at ambient temperature and humidity.


9.0 DISPOSAL

Product - This must be disposed of in compliance with anti-pollution and other laws of the country concerned. Contact your local service providers for further information.

Packaging - Disposal must be made in accordance with local waste management regulations. Outer packaging may be recycled. Contact your local service providers for further information.

10.0 SYMBOLS ON LABELLING

WARNINGS	
	Do not use if the pouch is damaged or opened.
	Medical Device
STORAGE	
	

	Title	Instructions for Use Barrier Cream	
	Revision	03	
	Technical Documentation Number	TF010	

11.0 PRESENTATION

Cat. No.	Size	No. / Box
BC2801	Barrier Cream 28g Tube	1
BC2812	Barrier Cream 28g Tube	12
BC9001	Barrier Cream 90g Tube	1
BC9012	Barrier Cream 90g Tube	12
BC0220	Barrier Cream 2g Sachet	20

12.0 COMPLAINTS


Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

For complaints, questions or comments, contact Avery Dennison Medical Customer Support at phone +353 43 3349586.



Avery Dennison Medical Ltd.
IDA Business Park, Ballinalee Road
Longford, N39 DX73. Ireland
phone +353 43 3349586
fax +353 43 3349566



Made in Ireland

	Title	Instructions for Use Barrier Cream	
	Revision	03	
	Technical Documentation Number	TF010	

13.0 REVISION HISTORY

Revision	Change History	Date
03	MDR release	30/09/2020

14. APPROVAL

Review and Approval	
<u>Name and Title</u>	<u>Signature and Date</u>
Elaine Minagh Regulatory Affairs Manager	 23/09/2020
Emmett McArdle R&D Manager	 30/09/2020