	Title	Instructions for Use Skin Cleanser (Spray/Stream Output)	
	Revision	Rev 04	
	Technical documentation Number	TF026	

1.0 PRODUCT DESCRIPTION

Skin Cleanser is a gentle, pH balanced no-rinse cleanser, designed to leave the skin supple and hydrated and can be easily dispensed with one hand.

2.0 INTENDED PURPOSE

Skin Cleanser is intended for incontinence and general cleansing needs.

2.1 INTENDED POPULATION

The product may be used on adults and children.

2.2 INTENDED USERS

Intended for use by health professionals and caregivers and may be used in a hospital, community and home setting.

3.0 INDICATIONS

Skin Cleanser indicated for incontinence and general cleansing needs. It is designed to be gentle on raw or sensitive skin associated with incontinence.

4.0 CONTRAINDICATIONS

None Listed.

5.0 WARNINGS

For external use only. Avoid contact with eyes. If contact occurs, rinse eyes thoroughly with water. Consult a physician if the condition worsens or persists for 7 days. Skin Cleanser should not be used on individuals who are known to have allergic sensitivity to any ingredients listed. Keep out of reach of children.

6.0 DIRECTIONS FOR USE


To use as a spray: Toggle the spray nozzle to the spray mode by twisting the spray nozzle to the spray setting. Depress the trigger to generate spray.

To use as a stream: Toggle the spray nozzle to the stream mode by twisting the spray nozzle to the stream setting. Depress the trigger to generate stream.

The Skin Cleanser should be applied directly onto the soiled area or onto a warm washcloth. Wipe clean, pat dry (no rinsing required). Repeat when necessary.

INGREDIENTS

Purified Water, Sodium Laureth Sulphate, Glycerine, Decyl Glucoside, Disodium EDTA, Cocoamidopropyl Betaine, Phenoxyethanol, Sodium Hydroxide, Ethylhexylglycerin.

	Title	Instructions for Use Skin Cleanser (Spray/Stream Output)	
	Revision	Rev 04	
	Technical documentation Number	TF026	

7.0 CLINICAL BENEFITS

Dressings can be removed easily from the skin, while the moisturiser present ensures conditioning of the skin.




8.0 STORAGE

Store away from direct sunlight at ambient temperature and humidity.

9.0 DISPOSAL

The Skin Cleanser can be disposed of based on the hospital or healthcare professional advice.

10.0 SYMBOLS ON LABELLING

WARNINGS	
	Caution
	Do not use if the pouch is damaged or opened.
STORAGE	
	

11.0 PRESENTATION

Skin Cleanser is available in a 235ml bottle with either a Spray / Foam or Spray / Stream for application to the skin.

12.0 COMPLAINTS

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.


For complaints, questions or comments, contact Avery Dennison Medical Customer Support at phone +353 43 3349586.

CE



Avery Dennison Medical Ltd.
IDA Business Park, Ballinalee Road
Longford, N39 DX73. Ireland
phone +353 43 3349586
fax +353 43 3349566

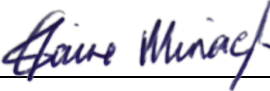

Made in Ireland

	Title	Instructions for Use Skin Cleanser (Spray/Stream Output)	
	Revision	Rev 04	
	Technical documentation Number	TF026	

13.0 REVISION HISTORY

Revision	Change History	Date
04	MDR Release	23/09/2020

14.0 APPROVAL

Review and Approval	
<u>Name and Title</u>	<u>Signature and Date</u>
Elaine Minagh Regulatory Affairs Manager	 23/09/2020
Emmett McArdle R&D Manager	 23 Sep 2020